



the **PortaSCC**[®] milk test

The PortaSCC™ milk test is designed as an on-farm management tool that can be used standalone or combined with other screening programs such as Rapid Mastitis Test (RMT), Conductivity or basic hand stripping.

It provides you with an effective way of screening for sub clinical mastitis when you need quick results.

WHEN CAN I USE THE TEST?

- Monitoring hot cows
- Checking at Calving and Drying off
- Monitoring sick cows after treatment
- Routine screening of cows in the herd
- Bulk tank SCC testing

HOW DOES THE TEST WORK?

- The test is based on a reaction between the cell-surface enzyme present in the milk and a colour producing chemical in the test strip. The darker the reaction colour, the higher the cell count

HOW DOES THE PORTASCC™ COMPARE TO A LAB TEST?

- In the lab and in field studies by major universities, the PortaSCC™ milk test shows good correlation with lab data

HOW FRESH SHOULD THE MILK BE?

- You have up to 8 hours to test the milk

WHAT IS THE RANGE OF THE DIGITAL TESTER?

- The range is 50,000 which is shown as LO, to 3 million which is shown as HI

Affordable and as easy as 1,2,3!



1

Add 1 drop milk



2

Add 3 drops activator



3

Use digital reader or compare with colour chart

PORTA CHECK SCC MILK TEST	CODE
Porta Check 24 test kit	NCI0250
Porta Check 100 test kit	NCI0255
Porta Check 500 test kit	NCI0260
Porta Check digital reader	NCI0265

